


CLEARsplint®

*Interim restorations
free of MMA*



for a perfect smile

astron®

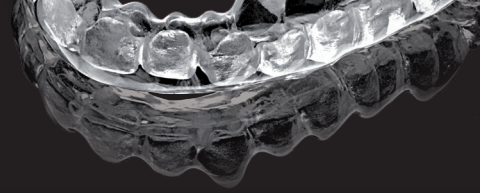

Advantages of CLEARsplint®:

- ☉ absolutely precise adaption
- ☉ self-adjusting
- ☉ hypoallergenic - free of MMA
- ☉ tension-free wearing comfort
no feeling of pressure in the mouth
- ☉ always transparent - will not yellow with age
- ☉ relinable and repairable

Important care instructions:

To avoid risk of breakage of your nightguard or partial denture it is absolutely necessary to put it - prior to each wearing - in approx. 50 °C warm water for at least 20 seconds!

Please do not use alcohol-based or commercial denture cleaners!

We recommend **astron® DuraClean²™**

Special cleaning agent for CLEARsplint® nightguards and interims



DuraClean²™

- Economical in use - enough for 48 cleanings
- cleaner (170 g) with box and brush REF 26170S
- cleaning powder - box (170 g) REF 26171
- cleaning brush REF 26172
- cleaning box REF 26173

EC REP

KENTZLER-KASCHNER DENTAL GmbH
Mühlgraben 36
73479 Ellwangen/Jagst
Telefon: +49 (7961) 90 73-0
Fax: +49 (7961) - 5 20 31
info@kkd-topdent.de
www.kkd-topdent.de



Astron Dental Corp.
Lake Zurich, Ill. 60047
Made in USA

CLEARsplint®

*Premium Nightguards
free of MMA*



The perfect Protection!

astron®

**KENTZLER
KASCHNER
DENTAL**

M260398/17.12.2020

astron[®] CLEARsplint[®]



A deformed CLEARsplint[®] comfort splint adapts easily and precisely to the teeth thanks to the thermoactive property of the material and its excellent flexibility. Even under heat, the perfect surface hardness is maintained.



Quality and aesthetics made by masters - presented by:



Anatomical model



...with exactly fitting bite protection



Advantages that speak for themselves! astron CLEARsplint[®] as nightguard:

- ⊖ thermoactive flexibility of the nightguard provides a tension-free wearing comfort
- ⊖ self-adjusting in the mouth, if you put it into warm water (40-50 °C) for activate the flexibility before use
- ⊖ hypoallergenic – free of MMA
- ⊖ better adaptation enables higher and pain-free wearing comfort - long wearable
- ⊖ transparent and optical clarity allows almost unnoticed use
- ⊖ repairs and corrections are possible
- ⊖ easy to remove for daily cleaning
- ⊖ extremely break resistant, hence optimal protection of the teeth

important patient information on the back --->

Advantages that speak for themselves! astron CLEARsplint[®] as denture:

- ⊖ aesthetic dentures free of metal clamps
- ⊖ visibility of the natural gums
- ⊖ transparent and optical clarity allows almost unnoticed use
- ⊖ the thermoactive flexibility of the interim restoration enables a precise adjustment - excellent clamping action and easy removal
- ⊖ self-adjusting in the mouth, if you put it into warm water (40-50 °C) for activate the flexibility before use
- ⊖ hypoallergenic – free of MMA
- ⊖ repairs, corrections and enlargements are possible
- ⊖ easy to remove for daily cleaning
- ⊖ high break resistance

important patient information on the back --->